

iglove

Part No: K-GLOVE/K-GLOVE-L

TensCare™



Conductive Garment for use with TensCare TENS & EMS units.

For single patient use only.

Two Gloves, supplied with adapter lead.

Supplied with one pack of two reference electrode pads (50 x 90 mm electrodes).



Manufactured by:

TensCare Ltd

PainAway House,
9 Blenheim Road,
Longmead Business Park,
Epsom, Surrey KT19 9BE UK.
Tel: +44 (0)13 72 72 34 34
Web: www.tenscare.co.uk

TensCare have altered the specification of the iGlove to improve performance:

- We have changed the fabric to a higher silver content to give improved contact.
- We now supply two gloves for use with each hand.
- We now supply self adhesive reference electrodes, to give a more reliable current distribution.

Instructions for Use

1. Ensure that the stimulator is turned **OFF**.
2. Insert one pin connector (from one lead pair) into the connector attached to the iGlove, and the other (from the same lead pair) into the connector on the electrode pad.
3. The gloves should be worn slightly damp to improve current flow.
4. Apply the garment
5. Place the electrode pad on the same limb as the iGlove.
6. If necessary, hand wash the iGlove and dry flat at room temperature.

CAUTION

If being used for pain management for Diabetic Neuropathy or other conditions where there is reduced or no sensory sensation then there is a small risk that reduced or uneven electrical contact may cause irritated or broken skin. Care needs to be taken to monitor the skin condition under the garment at frequent intervals.